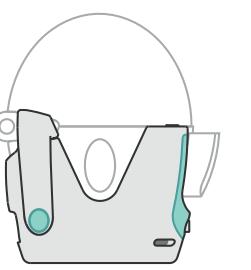
For use only with HandiHaler™

EN Product Manual

adherium

This Product Manual will help you to get the most out of your Hailie® sensor. Please read the manual before using the sensor.

Hailie® Sensor Intended Use

The Hailie® sensor is intended for single patient use in the home environment as a medication reminder and electronic data capture accessory for recording usage of prescribed inhaler medication. This may be used in the following applications:

- In clinical trials, where researchers need to know when a patient has used their trial inhaler medication.
- In clinical practice, where specialists, general practitioners, nurses, and educators need to know if a patient has used their prescribed inhaler medication.
- In self-management, where patients need to track their medication use as part of their management plan.

The Hailie® sensor is compatible only with the HandiHaler™ inhaler.

Important User Information

⚠ Warnings identify actions or situations that could lead to personal injury. Take note of all warnings before using the Hailie® sensor.

⚠ Precautions identify actions or situations that could damage the Hailie® sensor or other equipment, or affect the accuracy or availability of recorded usage data.

✓ Notes contain advisory information about some aspect of the Hailie® sensor or its use.

Warnings and Precautions

⚠ To ensure your Inhaler functions correctly, do not use the Hailie® sensor with any other Inhaler or medication than that indicated on the Hailie® sensor label.

⚠ Refer to the labelling provided with the Inhaler for instructions on use. Carry out all steps required to use your Inhaler according to the instructions.

⚠ The Hailie® sensor is intended to track medication usage. It is not intended to diagnose your condition or to replace the diagnosis of a licensed physician.

⚠ This manual does not provide information on how to use prescription medication, and is not intended to replace the advice provided by a health professional. Directions for using prescription medication should be obtained from a health professional and followed accordingly. Any questions related to prescription medication should be referred to a health professional.

⚠ US only: Read the label on the front panel of the Inhaler before installing into the Hailie® sensor.

⚠ US only: Important instructions for use of the Inhaler are printed on the sides of the inhaler. The labels are covered when the Hailie® sensor is installed. The labels can be checked at any time by removing the Hailie® sensor from the Inhaler, and replacing it after the label is checked. Follow the steps in *Installing or Removing an Inhaler*.

⚠ The Hailie® sensor does not contain a dose counter. Do not use data collected by the Hailie® sensor to determine the number of doses remaining in an Inhaler.

⚠ If your prescribed Inhaler changes, do not use it with this Hailie® sensor. Only use a Hailie® sensor that is labelled as compatible with your prescribed Inhaler.

⚠ Keep the Hailie® sensor outside MRI scanner rooms.

⚠ To ensure accuracy of inhaler usage records, use the inhaler within 5 minutes of opening the inhaler Dust Cap, and always close it again afterwards.

⚠ Take note of the following to prevent triggering a false usage record. Avoid using the sensor in excessively noisy environments. Do not handle the sensor excessively or slide it across a table, while the inhaler Dust Cap is open.

⚠ The Hailie® sensor will not monitor inhaler use for 5 seconds after the inhaler is installed into the sensor.

⚠ The Hailie® sensor is a battery-powered electronic device. Take care to not spill liquids on the Hailie® sensor or immerse it in water. Do not use the Hailie® sensor if it is not in good condition.

⚠ Do not expose the Hailie® sensor to excessive perspiration during exercise.

⚠ For hygiene and data integrity reasons, do not use the Hailie® sensor with more than one patient.

⚠ Remove the Hailie® sensor from the Inhaler before cleaning the Inhaler.

⚠ Do not use the Hailie® sensor while charging to avoid risk or electric shock.

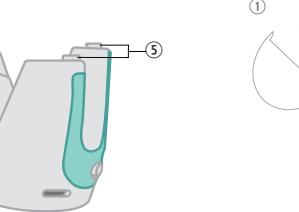
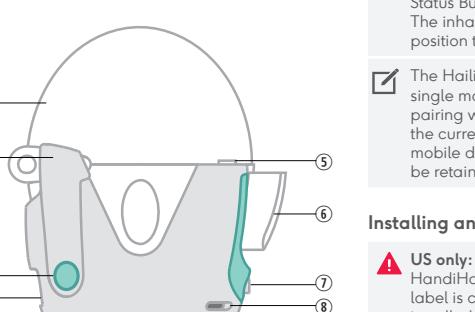
What is the Hailie® Sensor?

The Hailie® sensor is a companion tool that tracks your prescribed medication use and reminds you when to take your prescribed medication.

The Hailie® sensor is a small battery-powered electronic data logger that attaches to an inhaler. The Hailie® sensor contains an electronic clock and calendar that is used to log the date and time of inhaler usage. Usage data can be uploaded via a wireless Bluetooth® connection.

The Hailie® sensor has a built-in audiovisual alert to remind you when to take your prescribed medication — and tracks when you did — so you can have confidence you're following your prescription.

Hailie® Sensor Components With the HandiHaler™ installed

To insert the HandiHaler™, open the HandiHaler™ Dust Cap.

Key

1. HandiHaler™ Dust Cap
2. Clips
3. Release Buttons
4. USB Port
5. Retaining Fingers
6. HandiHaler™ Piercing Button
7. Status Button
8. LEDs (Light-emitting diode)

Compatible Inhalers

The Hailie® sensor is designed to work only with the HandiHaler™ inhaler as indicated on the Hailie® sensor label.

The Hailie® sensor intended population is identical to the intended population of the compatible inhaler.

Preparation for First Use

Following manufacture, the Hailie® sensor is placed in Deep-Sleep mode to conserve battery life. To check if the Hailie® sensor is still in Deep-Sleep mode, press the Status Button once. The LEDs will flash white if it is still in Deep-Sleep.

The Hailie® sensor must be paired with a compatible Bluetooth® mobile device in order to sync stored usage data.

Download a compatible mobile app to communicate with your sensor. For more information go to support.hailie.com.

Ensure Bluetooth® is enabled on your mobile device.

1. Before first use, place the Hailie® sensor on charge for up to 4 hours or until the LEDs stop flashing green.

2. When charging is completed, open the compatible mobile app and follow the prompts to complete the pairing process.

3. Disconnect the Hailie® sensor from the power supply. This will wake the Hailie® sensor from Deep-Sleep mode.

4. The Hailie® sensor will automatically enter Bluetooth® pairing mode and the LEDs will flash blue for 60 seconds.

5. The LEDs will flash green to indicate successful pairing.

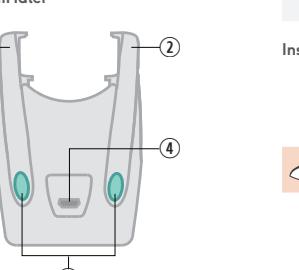
✓ If pairing is not successful within 60 seconds the LED will flash red. The pairing process can be repeated if necessary by pressing the Status Button 3 times quickly to initiate pairing. The inhaler Dust Cap must be in the closed position to enter pairing mode.

✓ The Hailie® sensor can only be paired with a single mobile device at one time. Successful pairing with a new mobile device will remove the current pairing. If pairing on the new mobile device fails, the existing pairing will be retained.

Installing and Removing an Inhaler

⚠ US only: Important instructions for use of the HandiHaler™ are printed on the inhaler. The label is covered when the Hailie® sensor is installed. The label can be checked at any time by removing the Hailie® sensor from the HandiHaler™, and replacing it after the label is checked.


Without the HandiHaler™

⚠ US only: Read the label on the sides of the inhaler before inserting into the Hailie® sensor.

⚠ To ensure accuracy of inhaler usage records, use the inhaler within 5 minutes of opening the inhaler Dust Cap, and always close it again afterwards.

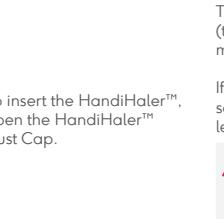⚠ Take note of the following to prevent triggering a false usage record. Avoid using the sensor in excessively noisy environments. Do not handle the sensor excessively or slide it across a table, while the inhaler Dust Cap is open.

To insert the HandiHaler™, open the HandiHaler™ Dust Cap.

Key

1. HandiHaler™ Dust Cap
2. Clips
3. Release Buttons
4. USB Port
5. Retaining Fingers
6. HandiHaler™ Piercing Button
7. Status Button
8. LEDs (Light-emitting diode)

Compatible Inhalers

The Hailie® sensor does not detect or record the quantity of medication delivered by the inhaler.

Cleaning the Hailie® Sensor and Inhaler

Check the instructions from the inhaler manufacturer for keeping the inhaler and mouthpiece clean.

⚠ Remove the Hailie® sensor from the inhaler before cleaning the inhaler.

Keep the Hailie® sensor clean and free of chemicals, steam, water and dust. Clean the outside plastic enclosure with a lightly dampened cloth. Check the Hailie® sensor is clean and repeat if necessary. Leave it dry in a warm place that is less than 30°C (86°F).

Medication Reminders

Audiovisual reminders may be set up from the Hailie® app. If on-sensor reminders are enabled, the medication reminders are generated on the Hailie® sensor with increasing frequency for up to 24 minutes, until either the prescribed medication dose is taken or the reminder is cancelled. The LEDs flash green and the Hailie® sensor emits a beeping sound when the reminder alert is triggered.

Reviewing Inhaler Usage Data

The inhaler usage history can be automatically uploaded via a compatible paired mobile device.

Storage

To help maintain battery capacity, store your Hailie® sensor below 30°C (86°F). Keep out of direct sunlight and avoid extreme temperatures.

Operating Temperature

The inhaler medication has its own storage recommendations. Refer to the labelling provided with the inhaler medication.

Troubleshooting

If the Hailie® sensor is not responding to Bluetooth® communications, check the Hailie® sensor battery level as per *Reviewing the Battery Level* in this manual. Ensure Bluetooth® is enabled in your mobile device.

Manually Uploading Stored Information

The Hailie® sensor can manually upload data to a paired Bluetooth® mobile device.

When travelling

ensure your mobile device is set to the local time zone for accuracy of inhaler usage data.

✓ When travelling ensure your mobile device is set to the local time zone for accuracy of inhaler usage data.

Removal

⚠ To ensure medication usage is detected, wait for 3 seconds after medication usage before removing the Hailie® sensor.

Reviewing the Battery Level

The LEDs on the Hailie® sensor indicate the battery level after detecting medication usage, or when the Status Button is pressed once.

LED Colour

Battery Status

Green The battery level is good. The Hailie® sensor is monitoring inhaler usage.

Orange The battery level is getting low. The Hailie® sensor is still monitoring inhaler usage.

Red The battery is depleted. The Hailie® sensor has stopped monitoring inhaler usage.

No Flash The battery is depleted, or the Hailie® sensor has failed, e.g. due to liquid immersion or mechanical stress.

Delivering a Dose of Medication

⚠ Refer to the labelling provided with the inhaler for instructions on use. Carry out all the steps required to use your inhaler according to the instructions.

The Hailie® sensor logs inhaler usage by monitoring the sound made by inhaling the medication, when the inhaler Dust Cap is opened and closed.

⚠ To ensure accuracy of inhaler usage records, use the inhaler within 5 minutes of opening the inhaler Dust Cap, and always close it again afterwards.

⚠ Take note of the following to prevent triggering a false usage record. Avoid using the sensor in excessively noisy environments. Do not handle the sensor excessively or slide it across a table, while the inhaler Dust Cap is open.

Disposing of the Hailie® Sensor

The Hailie® sensor contains a rechargeable battery that is recharged via the USB Port. To recharge the Hailie® sensor, connect it to a computer USB port or USB wall plug adaptor, using the supplied USB cable or a micro-USB cable less than 3 metres (9ft) in length.

EU only: Do not dispose of the Hailie® sensor as unsorted municipal waste. The Hailie® sensor must be recycled in accordance with Directives 2012/19/EU and 2006/68/EC. To arrange for return or disposal of the Hailie® sensor contact the supplier.

Warranty

The Hailie® sensor includes a 24 month warranty against manufacturing defects from date of first use. This warranty may be voided under the following circumstances: damage to the Hailie® sensor including dropping, water damage resulting from condensation or immersion, tampering, attempts to service, or other forms of abuse.

The Hailie® sensor LEDs flash green, orange or red (to indicate battery level) 3 seconds after detecting medication usage and the inhaler Dust Cap is closed.

If the LEDs flash red or if there is no flash, the Hailie® sensor is not monitoring usage. Check the battery level according to *Reviewing the Battery Level*.

⚠ This manual does not provide information on how to use prescription medication, and is not intended to replace the advice provided by a health professional. Directions for using prescription medication should be obtained from a health professional and followed accordingly. Any questions related to prescription medication should be referred to a health professional.

Up to 4 hours to fully recharge a Hailie® sensor

that is displaying red flashing LEDs. Charging times will significantly increase in temperatures below 10°C (50°F).

The Hailie® sensor should be recharged every 4 months as a minimum to ensure continued battery function.

Cleaning the Hailie® Sensor and Inhaler

Check the instructions from the inhaler manufacturer for keeping the inhaler and mouthpiece clean.

⚠ Remove the Hailie® sensor from the inhaler before cleaning the inhaler.

Keep the Hailie® sensor clean and free of chemicals, steam, water and dust. Clean the outside plastic enclosure with a lightly dampened cloth. Check the Hailie® sensor is clean and repeat if necessary. Leave it dry in a warm place that is less than 30°C (86°F).

The Hailie® sensor does not detect or record the quantity of medication delivered by the inhaler.

Medication Reminders

Auditory reminders may be set up from the Hailie® app. If on-sensor reminders are enabled, the medication reminders are generated on the Hailie® sensor with increasing frequency for up to 24 minutes, until either the prescribed medication dose is taken or the reminder is cancelled.

Reviewing Inhaler Usage Data

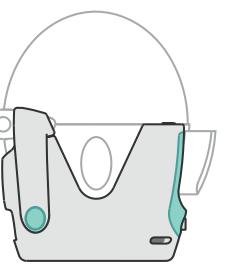
The inhaler usage history can be automatically uploaded via a compatible paired mobile device.

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